



CHEMICAL PERSONAL PROTECTIVE EQUIPMENT

Suit Support - Donning



EQUIPMENT: OUTER BOOTS, WATER, OUTER GLOVES, INNER GLOVES, MASK, GOGGLES, SUIT



REMOVE ALL JEWELRY AND PERSONAL EFFECTS



CAREFULLY INSPECT THE SEAMS OF THE SUIT FOR TEARS OR BROKEN SEALS



STEP INTO SUIT AND PULL UP TO WAIST LEVEL
THEN APPLY OUTER BOOTS



TAPE BOOTS ALONG UPPER RIM SEALING THE BOOT TO THE SUIT



CREATE A TAB ON THE TAPE'S END TO AID IN LATER DOFFING





CHEMICAL PERSONAL PROTECTIVE EQUIPMENT

Suit Support - Donning



REPEAT TAPING
PROCESS FOR
OTHER BOOT



APPLY
RESPIRATOR PER
THE MANUFACTURERS
DIRECTIONS



SEAL
CHECK THE
RESPIRATOR



APPLY EYE
PROTECTION



APPLY THE INNER GLOVES



APPLY THE
UPPER PORTION
OF THE SUIT.
MAKE SURE
SLEEVES
OVERLAP INNER
GLOVES





CHEMICAL PERSONAL PROTECTIVE EQUIPMENT

Suit Support - Donning

13



APPLY THE OUTER GLOVES

14



TAPE GLOVE ALONG UPPER RIM SEALING THE GLOVE TO THE SUIT

15



CREATE A TAB ON THE TAPE'S END TO AID IN LATER DOFFING

16



REPEAT TAPING PROCESS FOR OTHER GLOVE

17



REMOVE TAPE FROM FRONT ZIPPER FLAP OF SUIT

18



SEAL ZIPPER FLAP



CHEMICAL PERSONAL PROTECTIVE EQUIPMENT

Suit Support - Donning

19



PULL THE HOOD OF SUIT UP AND MAKE FINAL CHECK OF TAPE SEALS

REMOVING SUIT SUPPORT PPE

- Begin by cleaning any areas of the suit that may have been chemically contaminated if necessary.
- Carefully remove all tape from gloves and boots and unzip suit.
- Remove outer gloves being careful not to contaminate inner gloves.
- Remove suit only touching inner surfaces. Try to keep outer surfaces turned in on themselves.
- Step into a clean (cold zone) area.
- Once suit is removed, inner gloves can be removed and hands sanitized if desired.
- Remove respiratory protection last, then proceed to hand washing and shower.

NOTE: If any areas of your body were contaminated during the doffing process, decontaminate them well in the shower and **notify the decon team leader or appropriate personnel.**

