

# CHEMICAL PERSONAL PROTECTIVE EQUIPMENT

## Type C - Doffing - 3M



OPERATIONS  
TEAM CLEANS  
THE OUTSIDE  
OF THE SUIT  
BEFORE  
REMOVAL



MAINTAIN  
AIRWAY  
PROTECTION  
BY REMOVING  
PAPR HOOD  
TOWARDS END  
OF PROCESS



REMOVE  
PATIENT CARE  
GLOVES (IF  
WORN)



REMOVE TAPE  
SECURING  
OUTER  
PROTECTIVE  
GLOVES THEN  
REMOVE OUTER  
GLOVES



LOOSEN THE BELT WITHOUT REMOVING THE HOOD.



OPEN EXTERIOR  
ZIPPER SEAL

# CHEMICAL PERSONAL PROTECTIVE EQUIPMENT

## Type C - Doffing - 3M



07

REMOVE SUIT  
TOUCHING  
ONLY INTERNAL  
SURFACES

DOFFING  
PARTNER  
SHOULD  
ONLY TOUCH  
EXTERIOR



08

SLIDE THE SUIT DOWN SO THE PERSON CAN  
SIT.




09

REMOVE TAPE  
AT THE TOP OF  
THE OUTER  
BOOTS



10

REMOVE THE  
OUTER BOOTS

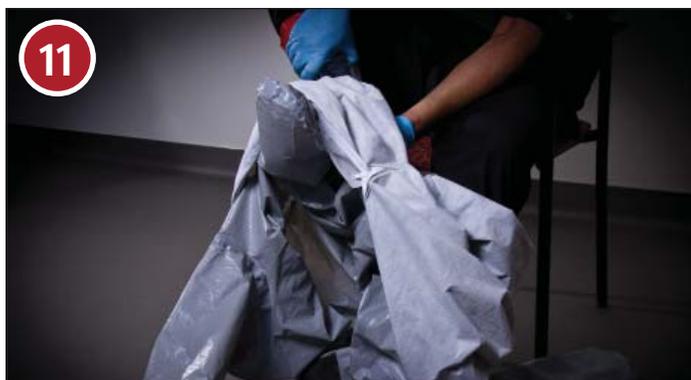

11

REMOVE THE LOWER PART OF THE SUIT



12

GATHER UP  
SUIT AND  
DISCARD

# CHEMICAL PERSONAL PROTECTIVE EQUIPMENT

## Type C - Doffing - 3M



13

GATHER UP HOOD AND SHROUD AND REMOVE OVER THE HEAD



14

REMOVE TAPE AT TOP OF INNER BOOT LINERS



15

REMOVE INNER BOOT LINERS



16

REMOVE INNER GLOVES USING GLOVE-IN-GLOVE TECHNIQUE



17

PERFORM HAND WASHING



18

DOFFING PARTNER TAKES AND RECORDS VITAL SIGNS

# CHEMICAL PERSONAL PROTECTIVE EQUIPMENT

Type C - Doffing - 3M



HYDRATE WITH  
DECAFFEINATED  
BEVERAGE



PROCEED TO  
PERSONAL  
SHOWER AND  
DEBRIEFING