



CHEMICAL PERSONAL PROTECTIVE EQUIPMENT

Type C - Doffing - 3M



01

OPERATIONS TEAM CLEANS THE OUTSIDE OF THE SUIT BEFORE REMOVAL



02

MAINTAIN AIRWAY PROTECTION BY REMOVING PAPR HOOD TOWARDS END OF PROCESS



03

REMOVE PATIENT CARE GLOVES (IF WORN)



04

REMOVE TAPE SECURING OUTER PROTECTIVE GLOVES THEN REMOVE OUTER GLOVES



05

LOOSEN THE BELT WITHOUT REMOVING THE HOOD.



06

OPEN EXTERIOR ZIPPER SEAL





CHEMICAL PERSONAL PROTECTIVE EQUIPMENT

Type C - Doffing - 3M



07

REMOVE SUIT TOUCHING ONLY INTERNAL SURFACES

DOFFING PARTNER SHOULD ONLY TOUCH EXTERIOR



08

SLIDE THE SUIT DOWN SO THE PERSON CAN SIT.



09

REMOVE TAPE AT THE TOP OF THE OUTER BOOTS



10

REMOVE THE OUTER BOOTS



11

REMOVE THE LOWER PART OF THE SUIT



12

GATHER UP SUIT AND DISCARD





CHEMICAL PERSONAL PROTECTIVE EQUIPMENT

Type C - Doffing - 3M



13

GATHER UP HOOD AND SHROUD AND REMOVE OVER THE HEAD



14

REMOVE TAPE AT TOP OF INNER BOOT LINERS



15

REMOVE INNER BOOT LINERS



16

REMOVE INNER GLOVES USING GLOVE-IN-GLOVE TECHNIQUE



17

PERFORM HAND WASHING



18

DOFFING PARTNER TAKES AND RECORDS VITAL SIGNS



CHEMICAL PERSONAL PROTECTIVE EQUIPMENT

Type C - Doffing - 3M

19



HYDRATE WITH
DECAFFEINATED
BEVERAGE

20



PROCEED TO
PERSONAL
SHOWER AND
DEBRIEFING

