



CHEMICAL PERSONAL PROTECTIVE EQUIPMENT

Type C - Donning - 3M

01

BATTERIES: YELLOW - RECHARGEABLE; WHITE - ONE TIME USE
EQUIPMENT: PAPR AND HOOD, BATTERIES, FILTERS, DUCT TAPE, SUIT, INNER BOOTS, OUTER BOOTS, INNER GLOVES, OUTER GLOVES

02

CAREFULLY INSPECT THE SEAMS OF THE SUIT FOR TEARS OR BROKEN SEALS

03

REMOVE CAPS FROM BOTH SIDE OF FILTER

04

ATTACH FILTERS TO PAPR

05

TEMPORARILY ATTACH BATTERY AND TEST PAPR MOTOR USING AIR FLOW METER

06

DONNING PARTNER TAKES AND RECORDS VITAL SIGNS





CHEMICAL PERSONAL PROTECTIVE EQUIPMENT

Type C - Donning - 3M



07

DRINK 1-2 GLASSES OF A NON-CAFEINATED BEVERAGE

REMOVE ALL JEWELRY AND PERSONAL EFFECTS



08

APPLY BOOT LINERS TO BOTH FEET AND TAPE TO OUTER SURFACE OF PANTS



09

CREATE A TAB ON THE TAPE'S END TO AID IN LATER DOFFING



10

STEP INTO SUIT AND PULL UP TO WAIST LEVEL



11

APPLY OUTER RUBBER BOOTS OVER SUIT. THE RUBBER GRIPS SHOULD BE ALONG THE BOTTOM TO IMPROVE TRACTION



12

TAPE BOOTS ALONG UPPER RIM SEALING THEM TO THE SUIT





CHEMICAL PERSONAL PROTECTIVE EQUIPMENT

Type C - Donning - 3M



13

CREATE A TAB ON THE TAPE'S END TO AID IN LATER DOFFING



14

APPLY THE INNER GLOVES



15

STAND AND APPLY THE UPPER PORTION OF THE SUIT



16

REMOVE AND SEAL THE LOWER PORTION OF THE ZIPPER TAPE



17

FRONT VIEW

REAR VIEW

APPLY PAPR MOTOR AND BELT. CLIP BATTERY TO BELT. ATTACH BATTERY TO PAPR MOTOR





CHEMICAL PERSONAL PROTECTIVE EQUIPMENT

Type C - Donning - 3M



18

APPLY THE OUTER GLOVES.

CAREFULLY TUCK THE CUFF OF THE SUIT INSIDE



19

TAPE GLOVE ALONG UPPER RIM SEALING THE GLOVE TO THE SUIT



20

CREATE A TAB ON THE TAPE'S END TO AID IN LATER DOFFING



21

APPLY PATIENT CARE GLOVES FOR ADDITIONAL DEXTERITY



22

ATTACH THE HOOD TO THE PAPR TUBING CONNECTED TO THE PAPR MOTOR



23

YELLOW BATTERY



WHITE BATTERY

TURN ON THE MOTOR AND PUT THE HOOD ON





CHEMICAL PERSONAL PROTECTIVE EQUIPMENT

Type C - Donning - 3M



24

TUCK THE INNER SHROUD INTO THE SUIT'S NECK



25

CLOSE THE ZIPPER ALL THE WAY TO THE NECK AND SEAL THE SUIT FLAP

FLATTEN OUTER FLAP OF HOOD OVER SHOULDERS



IMPORTANT

THE SUPPORT TEAM SHOULD BE AWARE OF HAND SIGNALS USED WHILE IN THE SUIT



PATting THE HEAD INDICATES NEEDING OUT OF THE SUIT



PUTTING HANDS AT THE NECK SIGNIFIES INABILITY TO BREATHE

