

CHEMICAL PERSONAL PROTECTIVE EQUIPMENT

Type C - Donning - ILC DOVER



EQUIPMENT: PAPR AND HOOD, BATTERIES, FILTERS, DUCT TAPE, SUIT, INNER BOOTS, OUTER BOOTS, INNER GLOVES, OUTER GLOVES



CAREFULLY INSPECT THE SEAMS OF THE SUIT FOR TEARS OR BROKEN SEALS



REMOVE CAPS FROM BOTH SIDES OF FILTER



ATTACH FILTERS TO PAPR



TEST PAPR MOTOR USING AIR FLOW METER



DONNING PARTNER TAKES AND RECORDS VITAL SIGNS

CHEMICAL PERSONAL PROTECTIVE EQUIPMENT

Type C - Donning - ILC DOVER



07

DRINK 1-2
GLASSES
OF A NON-
CAFEINATED
BEVERAGE

REMOVE ALL
JEWELRY AND
PERSONAL
EFFECTS



08

APPLY BOOT
LINERS TO
BOTH FEET AND
TAPE TO OUTER
SURFACE OF
PANTS



09

CREATE A
TAB ON THE
TAPE'S END TO
AID IN LATER
DOFFING



10

STEP INTO SUIT
AND PULL UP
TO WAIST LEVEL



11

APPLY OUTER RUBBER BOOTS OVER SUIT. THE
RUBBER GRIPS SHOULD BE ALONG THE BOTTOM TO
IMPROVE TRACTION






12

TAPE BOOTS
ALONG UPPER
RIM SEALING
THEM TO THE
SUIT

CHEMICAL PERSONAL PROTECTIVE EQUIPMENT


Type C - Donning - ILC DOVER

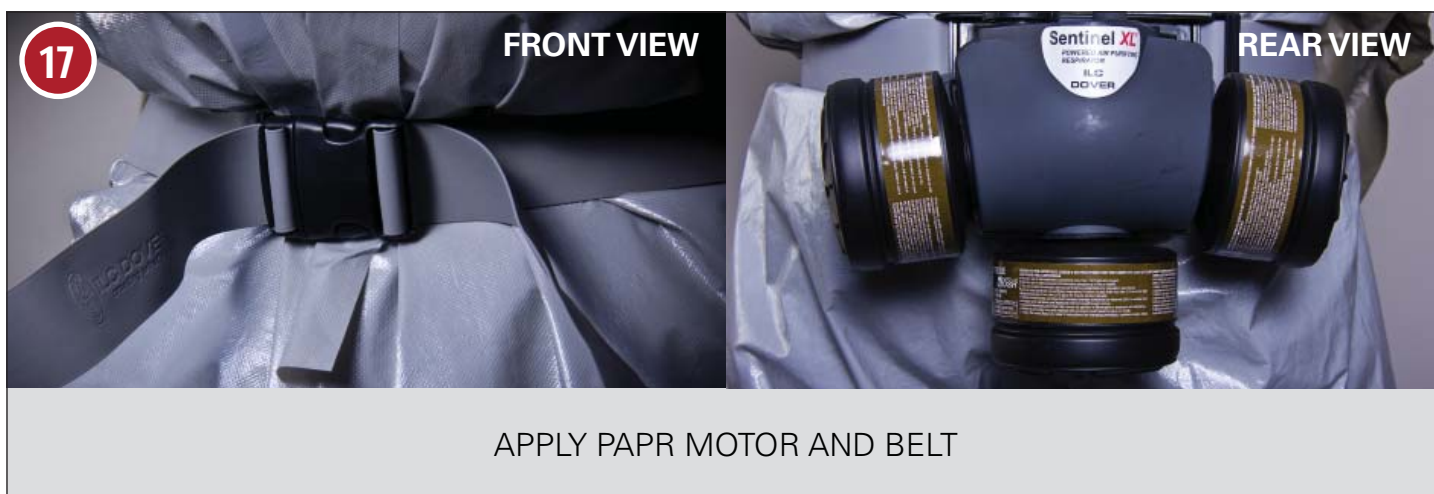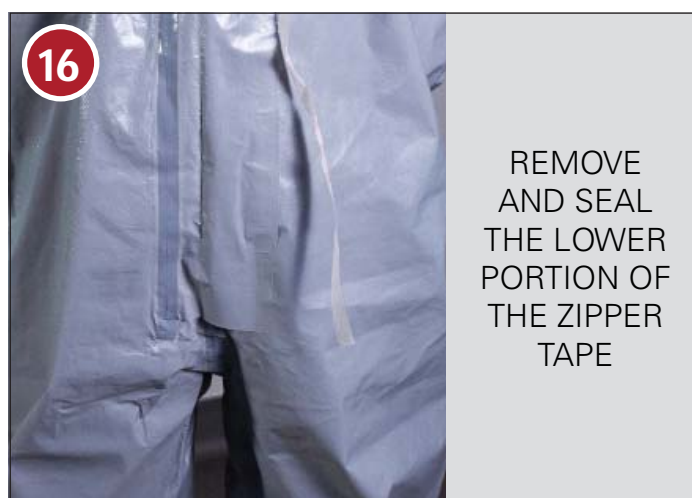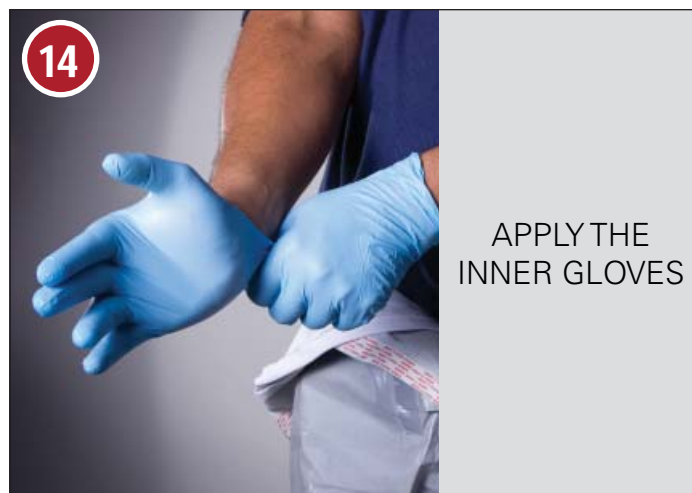

CHEMICAL PERSONAL PROTECTIVE EQUIPMENT

Type C - Donning - ILC DOVER

18



APPLY THE OUTER GLOVES.
CAREFULLY TUCK THE CUFF OF THE SUIT INSIDE

19



TAPE GLOVE ALONG UPPER RIM SEALING THE GLOVE TO THE SUIT

20



CREATE A TAB ON THE TAPE'S END TO AID IN LATER DOFFING

21



APPLY PATIENT CARE GLOVES FOR ADDITIONAL DEXTERITY

22



ATTACH THE HOOD TUBING TO THE PAPR MOTOR

23



TURN ON THE MOTOR AND PUT THE HOOD ON

CHEMICAL PERSONAL PROTECTIVE EQUIPMENT

Type C - Donning - ILC DOVER



24

TUCK THE
INNER SHROUD
INTO THE SUIT'S
NECK



25

CLOSE THE
ZIPPER ALL THE
WAY TO THE
NECK AND SEAL
THE SUIT FLAP

FLATTEN
OUTER FLAP
OF HOOD OVER
SHOULDERS



IMPORTANT

THE SUPPORT TEAM SHOULD BE AWARE OF HAND SIGNALS
USED WHILE IN THE SUIT



PATting
THE HEAD
INDICATES
NEEDING OUT
OF THE SUIT



PUTTING
HANDS AT THE
NECK SIGNIFIES
INABILITY TO
BREATHE